




# NIC Detect


ORAL COTININE TEST

## POSITIVE

## NEGATIVE

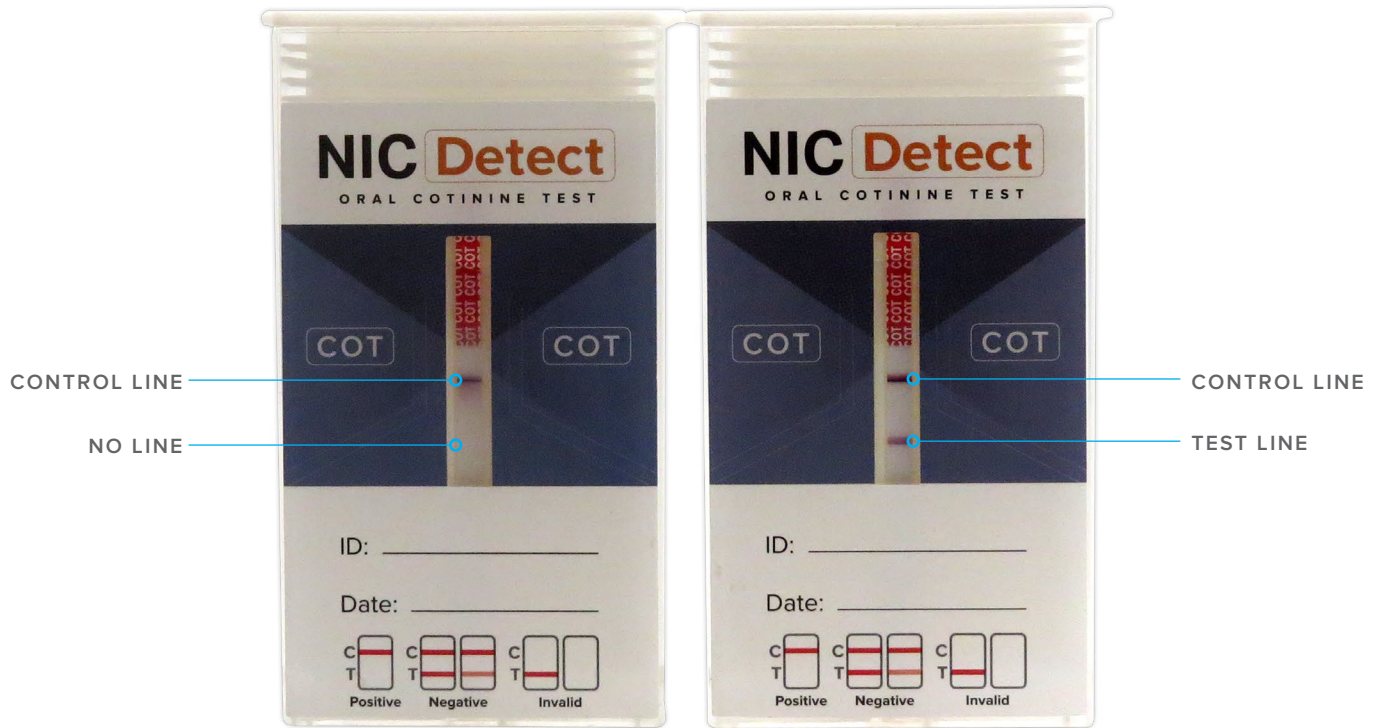

## INVALID

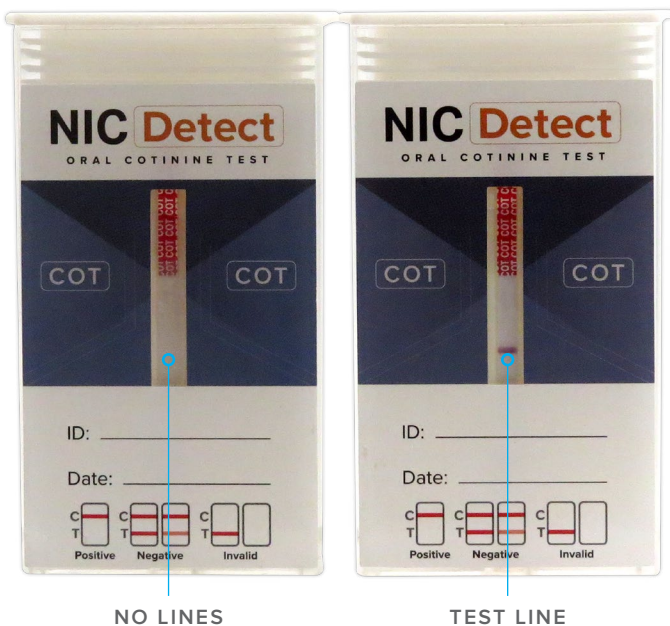

- **NEGATIVE:** \*All test lines appear. One colored line should be in the control region (C), and other apparent colored line should be adjacent in the test region (T). This negative result indicated that the drug concentration is below the detectable level or drug free.

\*NOTE: The shade of color in the test region (T) will vary, but it should be considered negative whenever there is even a faint colored line.

- **POSITIVE:** One colored line appears in the control region (C). Any test line not appears in the test region (T). This positive result indicates that the drug concentration is above the detectable level.
- **INVALID:** Control line fails to appear. Insufficient specimen volume or incorrect procedural techniques are the most likely reasons for control line failure. Review the procedure and repeat the test using a new test panel. If the problem persists, discontinue using the lot immediately and contact the manufacturer.