



Instructions For Use for Saliva Confirmation Service

The results obtained from instant result drug tests are preliminary (qualitative) in nature and your decision to purchase a confirmation test will allow you to receive evidential (quantitative) results. Please do not send the Instant saliva device to the lab as they WILL NOT accept instant devices. An additional sample must be collected using the SalivaBio device in order to be confirmed at the lab. Please make sure to follow the directions below in order to have your saliva sample processed. The saliva sample sent to the laboratory will provide quantitative results for either the 65 analytes or 96 analytes purchased. Please follow these instructions to ensure that your sample is received in the proper condition at the laboratory for testing.

Included in your confirmation kit are the following materials:

SalivaBio Collection Aid & Cryogenic Vial along with package insert
Chain of Custody Form (CCF), Specimen Transport Bag, FedEx Clinical Pak
FedEx Shipping label

1. If you have used an instant saliva device you must collect a different saliva sample by using the SalivaBio collection Aid & Cryogenic Vial provided in your kit, please follow the SalivaBio Package Insert provided for proper collection and storage of saliva.
2. Peel the barcode from the CCF form and apply it to the SalivaBio collection device you would like to confirm. This will along with the CCF will ensure that the results of the laboratory testing match with the device you are sending.
3. Fill out the CCF with the name of the person providing their sample, as well as the collector's name, the date, and any additional information that is important. The 65 Analyte Saliva confirmation will be preselected and will be tested unless you purchased a 96 Analyte panel in which case you will cross out the 65 Analyte Saliva and write 96 Analyte Saliva directly below
4. Place the SalivaBio collection device in the specimen bag you have been provided and tape it securely shut.
5. Place the Specimen Bag along with the CCF in the FedEx Clinical Pak for shipping. Seal the bag.
6. Place the right side of the pre-addressed shipping label (the sticker) on the outside of the Clinical Pak.
7. Deliver the package to the nearest FedEx shipping location.
8. Please allow up to 72 hours from the time the sample has arrived at the laboratory to receive your results.

If you have any questions please contact:

Client Services
Tel: 866.465.2855
Or email us
Orders@drugtestsinbulk.com