



Instructions For Use for Urine Confirmation Service

The results obtained from instant result drug tests are preliminary (qualitative) in nature and your decision to purchase a confirmation test will allow you to receive evidential (quantitative) results. The urine sample sent to the laboratory will provide quantitative results for the drugs selected-whether illicit or prescription. Please follow these basic instructions to ensure that your sample is received in the proper condition at the laboratory for testing.

Included in your confirmation kit are the following materials:

Sample Security Box

Urine Monovette & User Guide

Chain of Custody Form (CCF) Specimen Transport Bag FedEx Clinical Pak FedEx

Shipping label

1. If you have used an instant result urine drug test cup, make certain that the lid on the cup is screwed on tightly (be careful not to over-tighten) so it will not leak during shipment.
2. If you are using a dip card or would prefer to send the specimen using the Urine Monovette we have included one in your shipment. The urine Monovette is less likely to spill, leak, or crack during shipment therefore you are more likely to NOT receive a quantity not sufficient from the lab. **We will not provide a credit for Quantity not sufficient samples.**
3. Peel the barcode from the CCF form and apply it on the lid of the test cup or collection device you would like to confirm. This will along with the CCF will ensure that the results of the laboratory testing match with the device you are sending.
4. Fill out the CCF with the name of the person providing their sample, as well as the collector's name, the date, and any additional information that is important. Select the drugs you would like confirmed by circling the presumptive positive drug (s) on your testing device.
5. Place the instant test device or collection device in the box you have been provided and tape it securely shut.
6. Place the sealed box along with the CCF inside the FedEx Clinical Pak for shipping. Seal the bag.
7. Place the right side of the pre-addressed shipping label (the sticker) on the outside of the Clinical Pak.
8. Deliver the package to the nearest FedEx shipping location.
9. Please allow up to 72 hours from the time the sample has arrived at the laboratory to receive your results.

If you have any questions please contact:

Client Services

Tel: 866.465.2855

Or email us

Orders@drugtestsinbulk.com